



Installation Instruction of RH-QIM03

Step 1. Take out all the components from the package

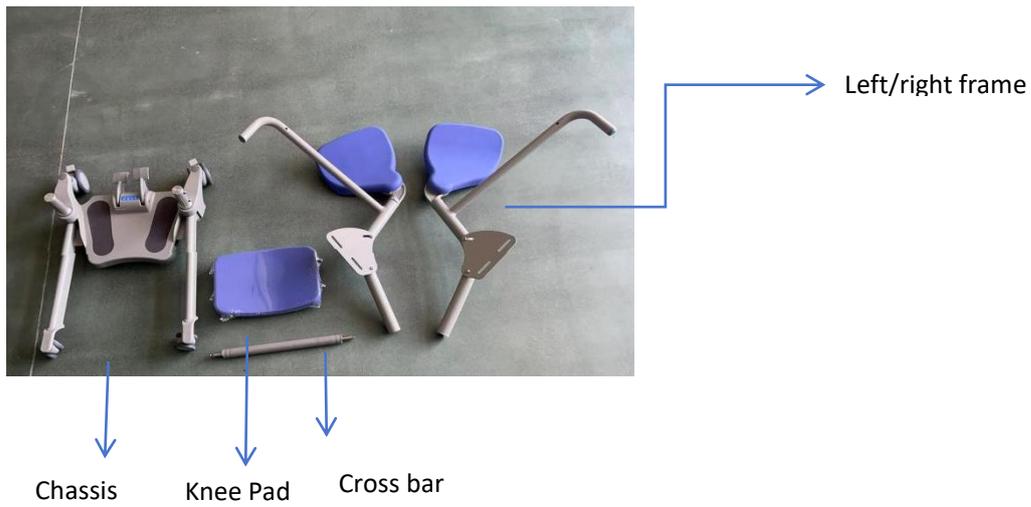

Step 2. Remove the screws at the circled mounting post on the chassis

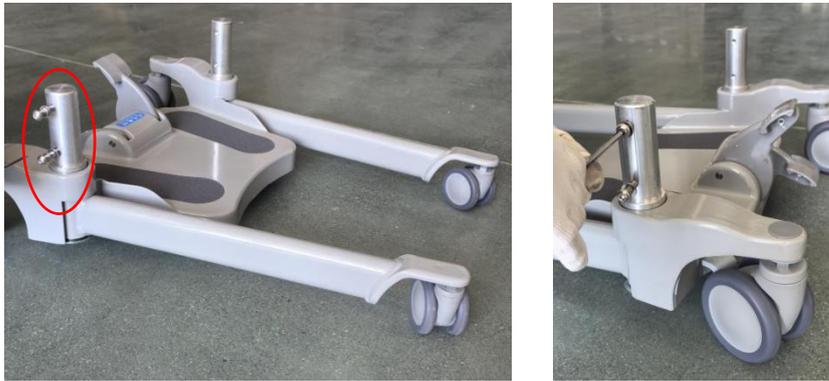


Step 3. Put the left/right frame onto the mounting post, align the holes and screw on



Step 4. Remove the screws and washers on the knee pad



Step 5. Black washers go between the metal plate and the pad, align the screw hole

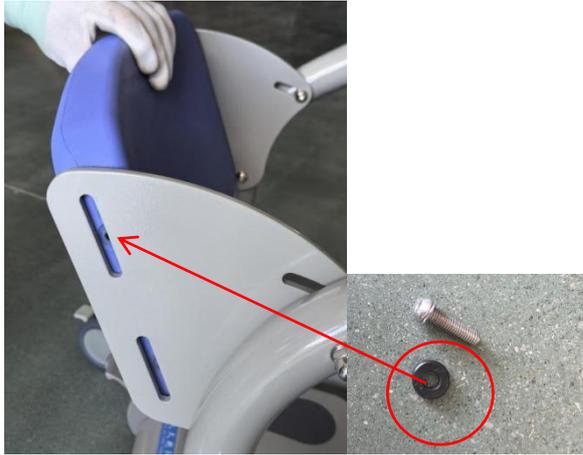

Step 6. Align the metal washer and the screw with the screw hole, screw up



Step 7. Screw off the bolt at the cross bar end, put the bar between the two frames and tighten the bolt from the outside.

